K-BLEACH CLINIC

ES	Kit de blanqueamiento dental profesional Peróxido de Hidrógeno 35% con Hidroxiapatita
EN	Professional teeth whitening kit 35% Hydrogen Peroxide with Hydroxyapatite
PT	Kit de clareamento dental profissional Peróxido de hidrogénio 35% com Hidroxiapatite

INSTRUCCIONES GENERALES

ES

DESCRIPCIÓN

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EN

DESCRIPTION

K-BLEACH CLINIC whitening is for professional in-clinic use only. The dual-syringe format ensures easy application and more precise dosing. This syringe is composed of two independent channels: one contains Hydroxyapatite and the other, Hydrogen Peroxide. Please read and understand the instructions carefully before use.

COMPOSITION

Aqua, Hydrogen Peroxide, Triethanolamine, Hydroxypropyl Methylcellulose, Hydroxyapatite.

SPECIFICATIONS

Pre-treatment process	Leave dual syringe at room temperature for 50 minutes before treatment
Cycle duration	8 to 15 minutes
VITA Shade Guide Order	B1, A1, B2, D2, A2, C1, C2, D4, A3, D3, B3, A3.5, B4, C3, A4, C4.
Gel layer thickness	2-3 millimeters

INTENDED PURPOSE

K-BLEACH CLINIC is intended exclusively for in-clinic use by dental professionals to whiten one or more teeth. It offers a conservative alternative method for treating dark or discolored teeth caused by various factors.

Benefits of Hydroxyapatite:

Hydroxyapatite is a mineral that makes up to 97% of dental enamel and around 70% of dentin. It provides several key benefits for oral health:

Main benefits of Hydroxyapatite for teeth:

- Enamel remineralization: Repairs micro-cracks and damage caused by daily wear and acidity.
- Protective coating: Forms a protective layer on the tooth surface, strengthening its structure.
- Natural whitening: Fills in enamel imperfections, making teeth appear whiter and shinier.
- Cavity prevention: Acts as a barrier against acid-producing bacteria.
- pH neutralization: Reduces enamel demineralization by balancing oral pH.

- Reduces sensitivity: Seals exposed dentin tubules, blocking external stimuli (cold, heat, sweets) from reaching nerves.
- Mild antibacterial properties: Reduces plaque adherence without disrupting the natural oral flora.
- Natural alternative to fluoride: A safe, effective option for those avoiding fluoride.
- Biocompatible and non-toxic: As a natural component of teeth and bones, it causes no irritation or long-term adverse effects.

INDICATIONS FOR USE

- 1- When used correctly, K-BLEACH CLINIC whitens most teeth with noticeable results. Avoid making unrealistic promises, especially in tetracycline-stained cases, where results may appear after 2+ sessions (in-clinic and at-home).
- 2- Although no data suggests damage to teeth, patients should be informed of possible temporary sensitivity, which typically resolves within 2 days.
- 3- Perform dental cleaning one week prior to treatment.
- 4- Cavities and other treatments must be addressed before whitening. In visible teeth, restorations should be provisional. Definitive fillings should be placed 20 days after whitening using the final tooth shade.
- 5- Gingival protection is essential to avoid irritation. Apply a 1 mm layer of gingival barrier above the gum margin, including worn incisal edges and diastemas.
- 6- Keep syringes refrigerated. Let sit at room temperature for 50 minutes before use.
- 7- If the gel contacts soft tissue, rinse thoroughly with water and apply vitamin E to the affected area.

INSTRUCTIONS

Perform professional cleaning one week before treatment.

1. Examine teeth and record initial shade using the VITA guide in the following order: B1, A1, B2, D2, A2, C1, C2, D4, A3, D3, B3, A3.5, B4, C3, A4, C4.
2. Take a photo of the initial shade.
3. Insert the cheek/lip retractor.
4. Polish teeth with the included pumice paste to remove impurities. Never use pastes with glycerin, gums, or oils.
5. Dry the teeth and gums using compressed air, then apply the gingival barrier, fully covering the tissue. Cure with light.
Caution: Overexposure to light may overheat the gums.
6. Attach the applicator tip to the dual syringe, discard the first mixed portion. Apply a 2-3 mm layer of gel directly to teeth from 4 to 4 / 5 to 5, upper and lower arch.
7. If using a whitening lamp, position the lamp head less than 2 centimeters from the lips. Set the timer and allow the product to activate for a cycle of 8 to 15 minutes.
If a dual-arch system is not used, start the cycle with the curing lamp approximately 6-7 millimeters from the tooth surface, moving it from tooth to tooth every 3 seconds until the cycle is complete.
8. After the first cycle, carefully suction the gel from each tooth. Reapply the gel and re-expose to light.
The gingival barrier can remain in place—no need to remove or rinse between cycles.
9. After the second cycle, suction the gel again, without rinsing. If the result is not satisfactory, perform a third identical cycle.
10. After the final cycle, rinse and remove the gingival barrier.
11. Check and record the new tooth shade with a photograph.
12. Provide syringes and trays for at-home maintenance. Suggested minimum: 2 syringes over 2 weeks.
13. Follow up the next day to assess shade and patient sensitivity. If results are satisfactory, recheck in 6 months.
If not, continue weekly syringe-at-home protocol (max 8-10 weeks).

FOODS TO AVOID DURING AND 72 HOURS AFTER TREATMENT

Tobacco, coffee, tea, soft drinks, red wine, rosé wine, grape juice, artichokes, beets, cherries, strawberries, blackberries, raspberries, pomegranates, watermelon, red grapes, chocolate, cocoa, carrots, broccoli, spinach, fried tomato, squid ink, stews with colorants / saffron, or any food with strong pigmentation.

CLINICAL INDICATIONS

- Internal whitening
- External whitening
- Whitening of endodontically treated teeth

RESTRICTIONS ON COMBINATIONS

K-BLEACH CLINIC must not be used in combination with other products, as this may affect whitening results.

INTENDED USER

K-BLEACH CLINIC is designed exclusively for use by dental professionals.

STERILITY

K-BLEACH CLINIC does not require sterilization, cleaning, or preparatory disinfection for safe and proper use.

STORAGE

Keep refrigerated. Let sit at room temperature for 50 minutes before use. Each syringe is intended for single use per patient. Do not reuse.

WARNINGS AND PRECAUTIONS

For exclusive distribution and use by dentists.
Do not ingest.
Prolonged use of peroxide whitening treatments is not recommended.
Not recommended for regular alcohol or tobacco users.
Do not use on pregnant women or children.
Do not use in patients under 18 years old.
Not recommended after recent dental restorations.
Do not use on patients with existing gum tissue lesions.
Avoid contact with eyes. IF CONTACT OCCURS: Rinse thoroughly and seek medical advice. Remove contact lenses if easy to do, and continue rinsing.
Wash hands thoroughly after handling. Use gloves, protective clothing, goggles, and mask.
Causes skin irritation. Causes serious eye damage. May irritate respiratory system.
IF SKIN CONTACT OCCURS: Wash with soap and water.
Avoid inhalation of dust / smoke / gas / mist / vapors / spray.
IF INHALED: Move victim outdoors and keep them at rest in a comfortable breathing position.
Dispose of contents in accordance with local regulations.
Contains hydrogen peroxide.

In case of poisoning or accidental ingestion, contact the Toxicology Information Service: +34 91 562 04 20

SHELF LIFE

The product has a shelf life of 2 years from the date of manufacture. Do not use after the expiration date indicated on the kit. Check the lot number and expiration date.

STORAGE CONDITIONS

Keep refrigerated.
Store away from heat sources, open flames, and out of reach of children. Store in its original packaging.

DISPOSAL

Waste and packaging must be handled and disposed of in accordance with applicable local and national regulations, complying with regulatory requirements.

MANUFACTURER'S RESPONSIBILITY

K-BLEACH CLINIC is safe and effective when used as directed in the manufacturer's instructions for use. This product is developed exclusively for professional dental use.

AVAILABLE FORMATS

K-BLEACH CLINIC is available in 2 formats:

K-BLEACH CLINIC - 1 Patient

Contents:

1 Dual syringe of 4 ml with 35% Hydrogen Peroxide and Hydroxyapatite
1 Protective resin (1.2 g)
1 Applicator brush
1 Vitamin E swab
1 Initial polishing paste (2 g)
Application tips
Instruction manual

K-BLEACH CLINIC - 3 Patients

Contents:

3 Dual syringes of 4 ml with 35% Hydrogen Peroxide and Hydroxyapatite
3 Protective resins (1.2 g)
3 Applicator brushes
3 Vitamin E swabs
3 Initial polishing pastes (2 g)
Application tips
Instruction manual

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